# MiniStretcher

SystemRoMedic™

User manual – English

Bruksanvisning – Svenska

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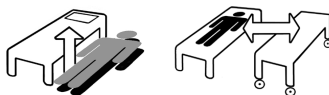
**REF**

4017

**SWL: 200kg/440 lbs**

## Table of contents

<b>English</b> .....	3-4
<b>Svenska</b> .....	5-6
<b>Norsk</b> .....	7-8
<b>Dansk</b> .....	9-10
<b>Suomi</b> .....	11-12
<b>Deutsch</b> .....	13-14
<b>Nederlands</b> .....	15-16
<b>Français</b> .....	17-18
<b>Italiano</b> .....	19-20
<b>Español</b> .....	21-22
<b>Product information</b> .....	23

MiniStretcher is used in emergency care units and ambulance services for lifting and transfer. Transfers between two locations, for example, bed to stretcher or lifting from the floor. MiniStretcher is easy to keep clean and store.

## Check Safety

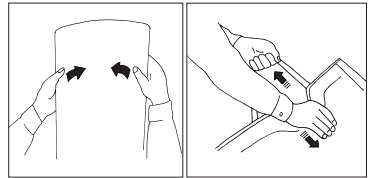
---

### Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded

### Stability test

Carefully bend the assistive device to check strength and durability. The board should be both elastic and rigid. Discard the assistive device when its strength and durability are diminished.

### Mechanical load

Apply pressure to buckles and handles and verify that they will not open or break under heavy loads.

## Always read the user manual

---

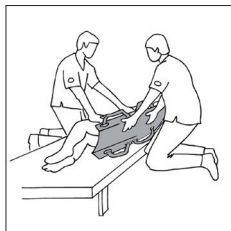
Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.


Always make sure that you have the right version of the user manual.

The most recent editions of user manuals are available for downloading from our website, [www.handicare.com](http://www.handicare.com).

## Placement and use of MiniStretcher

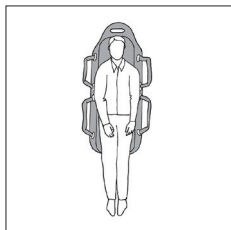Turn the user on one side and fold the handles under the MiniStretcher before it is pushed in under the user.

Place the user centred on the MiniStretcher and ensure that there is sufficient support for the head.

Work in pairs (or more) one on either side, or one at each end, and lift the user. Plan the transfer so as to minimize time taken, lifting and distance.

## Storage and cleaning

Keep the MiniStretcher on an ambulance stretcher, as shown in the picture.

MiniStretcher should be cleaned with a pressure washer and disinfectant.

### Material

Polyester

### Care of the product

Read the product label.

**CE** Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See [www.handicare.com](http://www.handicare.com) for a complete list of distributors.

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## Simple solutions for great results

SystemRoMedic™ is the name of Handicare's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic™ is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

## Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic™ is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic™ offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

## Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website; [www.handicare.com](http://www.handicare.com).

## For questions about the product and its use

Please contact your local Handicare and SystemRoMedic™ representative. A complete list of all our partners with their contact details can be found on our website; [www.handicare.com](http://www.handicare.com).

***Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.***

*The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes various manual and power wheelchairs, seating systems, scooters, a complete easy transfer system and other patient handling aids, stairlifts, car adaptations, rise and recline chairs, and bathing and toileting products.*

**handicare**

Handicare AB  
Maskinvägen 17  
SE-972 54 Luleå, SWEDEN  
Tel: +46 (0)8-557 62 200  
Fax: +46 (0)8-557 62 299  
E-mail: [info@handicare.se](mailto:info@handicare.se)  
[www.handicare.com](http://www.handicare.com)